92-U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM
92-U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM

U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM

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U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM

FOR COLD-WELDED AND BROKEN OFF
SCREWS

A situation you have experienced time and time again:

Everything seems well-prepared for the removal of an unspecified dual rod system. But then, you realize there is no fitting screwdriver for the set screw or that the tulip-head pedicle screw does not match your screwdriver... Thanks to our UNIVERSAL EASY SCREW REMOVAL SYSTEM, this situation will not create an unpredictable delay that could postpone your subsequent interventions - regardless of whether you are dealing with intact or broken pedicle screws or with older systems that still employed Schanz screws. With our UNIVERSAL EASY SCREW REMOVAL SYSTEM, you have a firm grip even on the most complicated explantations.

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92-U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM

NO NEED TO WORRY ABOUT COLD-WELDED, STRIPPED OR BROKEN OFF SCREWS

Who is not familiar with the following situation? Just half an hour of surgery time is calculated for removing the plate, and then ...

there is a cold-welded screw that cannot be undone or a screw with the head twisted off. Thanks to our 92-U-ESR system, the situation described will not turn into an incalculable time factor that makes later surgery steps last into the evening. No matter whether the screws are intact, broken off, cold welded or worn, you can handle even complicated screw removal with the 92-U-ESR system.

SAFE AND EASY

With our 92-U-ESR system, not only do you have the right extractor on hand at all times for the screw that needs removing - you also have a system that is truly easy and safe to use. The cases where other systems fail to remove a cold-welded screw are where you start getting warmed up with the 92-U-ESR system. The extremely robust instrument fits perfectly in your hand and enables you to remove even extremely jammed screws.

NEW DESIGN, NEW MATERIALS

Thanks to a new material mixture, the extractors are extremely durable and tough. The new design of the conical extractors for removing intact, worn or cold-welded screws makes it even easier to screw them into a screw head or the remainder of a screw head, joining very firmly with the entire screw in the process. The funnel-shaped design of the extractors for removing broken-off screws makes it possible to extract screws in a single step. It simply cannot be faster and easier than that.

ADVANTAGES OF THE U-ESR // UNIVERSAL EASY SCREW REMOVAL SYSTEM

- 12 different sizes for optimal adjustment.
- Not a single use product! It is possible to check intraoperatively which extractor fits best without incurring costs.
- Handle with ratchet function for comfortable working.
- Eccentric turn-out bar for very tight screws.
- Reduces costs and working time.

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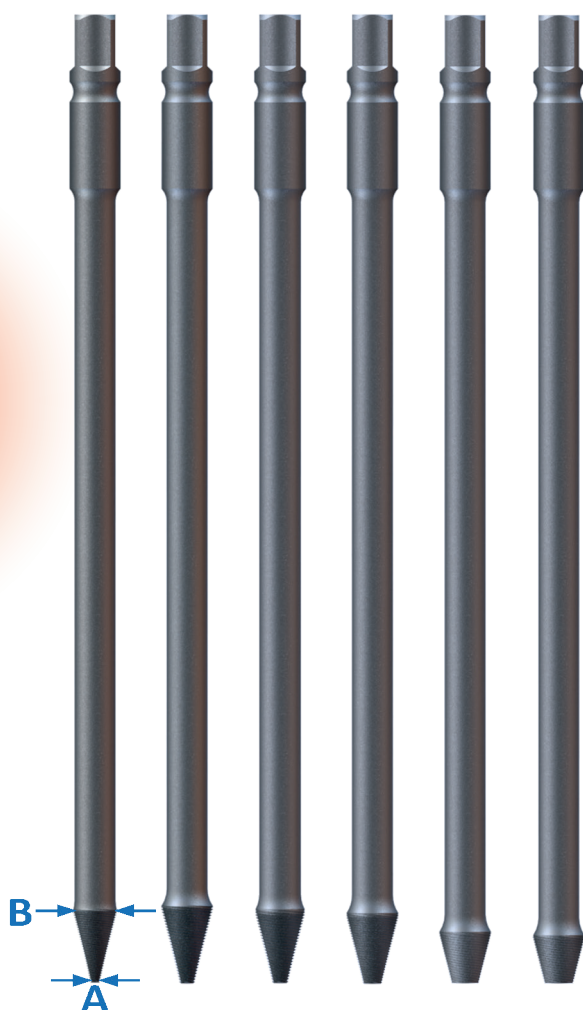
92-U-ESR KS-EXTRACTORS // EXTRA LONG

For minimally invasive screw removal.

92-U-ESR-KS extractors for stripped and cold-welded screw heads

Art.Nr. Extraktor Ref.No. Extractor	Konisches Außengewinde A Conical external thread A	Extraktorlänge Extractor length	Passend für Schraubenköpfe (Ø mm) Suitable for screw threads (Ø mm)
92-U-ESR-KS 1	A Ø 1.8 mm bis B Ø 8.5 mm	150 mm	TX. 4.0; TX. 5.0; TX. 6.0; HEX. 1.5
92-U-ESR-KS 2	A Ø 2.5 mm bis B Ø 7.5 mm	150 mm	TX. 7.0; TX. 8.0; HEX. 1.5; HEX. 2.5
92-U-ESR-KS 3	A Ø 3.1 mm bis B Ø 8.0 mm	150 mm	TX. 10; TX. 15; HEX. 2.5
92-U-ESR-KS 4	A Ø 3.5 mm bis B Ø 8.0 mm	150 mm	TX. 20; TX. 25; HEX. 3.0; HEX. 3.5; HEX. 4.0
92-U-ESR-KS 5	A Ø 4.0 mm bis B Ø 8.0 mm	150 mm	TX. 25; TX. 30; HEX. 4.0; HEX. 4.5
92-U-ESR-KS 6	A Ø 4.5 mm bis B Ø 8.0 mm	150 mm	TX. 25; TX. 30; HEX. 4.0; HEX. 4.5; HEX. 5.0

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IMPORTANT!

Extractors must always be turned counter-clockwise.

Please check the extractors for damage after use.

Extractors with damaged threads or teeth after use must be replaced.

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KS EXTRACTORS APPLICATION GUIDE

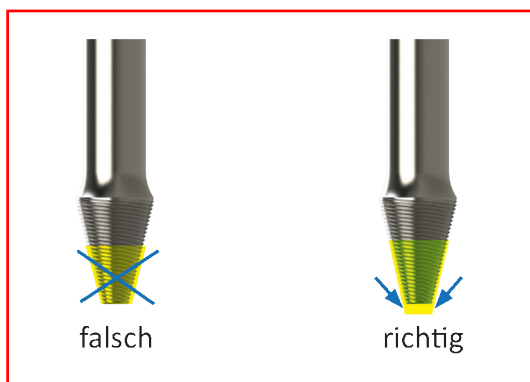
Extractors for stripped or cold-welded screw heads

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DETERMINING THE RIGHT KS EXTRACTOR:

Correctly selecting the appropriate extractor is of great importance for successful removal of the screws. Each extractor features a size specification that makes the initial selection process easier. The size specification refers to the diameter of the screw to be removed. The „KS“ extractors work like taps. In other words, the side flanks of the extractors are firmly screwed into the heads of the screw. For this principle to work properly, it is important to choose the size of the extractor so that the tip of the extractor does not touch the bottom of the screw head first, but only the flanks of the screw head. (Fig.)

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Always keep in mind that the available material necessary for the extractor to have sufficient hold in the screw head is always limited when removing stripped or cold-welded screws.

If you screw-in a KS extractor with the handle, there may be some tilting movements as you reposition your hand, which will re-loosen the first screwed threads.

Therefore, you should always use a drill with a 3-jaw chuck for screwing in the extractor. You can hold the drill tilt-proof while slowly screwing into the screw head.

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IMPORTANT!

Always turn extractors only to the left when screwing them in!

Starting from a certain torque, the drill will stop, because the force is too large. Remove the drill adapter and use the cross bar to finally release the screw from its position.

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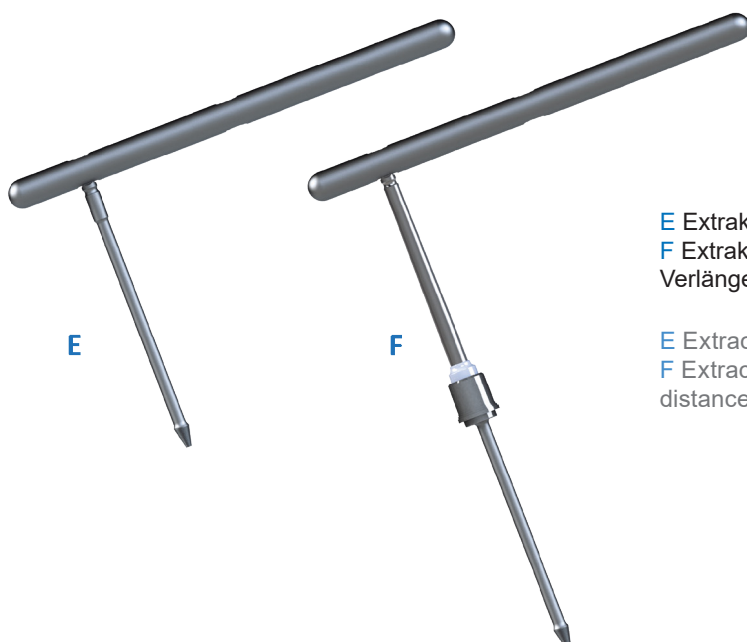
INSTRUMENTS AND KS EXTRACTORS APPLICATION EXAMPLES

The extractors can be combined with all instruments.

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- A Extractors in conjunction with the handles
- B Extractors in conjunction with the handles and distance adapter
- C Extractors in conjunction with the drill adapter
- D Extractors in conjunction with the drill adapter and distance adapter

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- E Extractors in conjunction with the turn-out bar
- F Extractors in conjunction with the turn-out bar and the distance adapter

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
92-U-ESR AS-EXTRACTORS // EXTRA LONG

For minimally invasive screw removal

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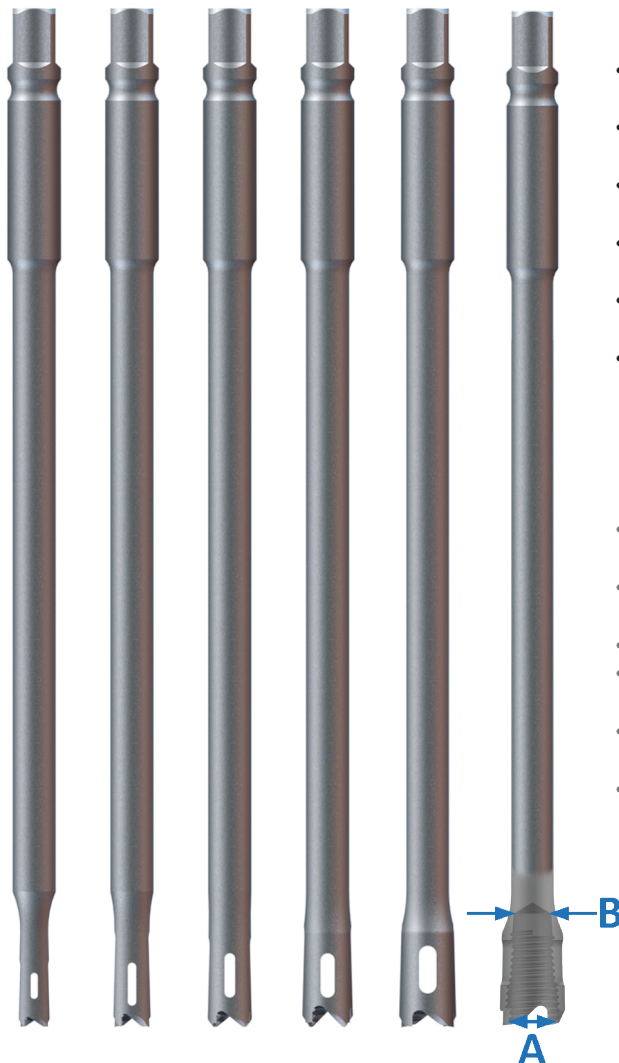- The new „one step“ extractor generation for the removal of broken off screws.
- The new design allows the screws to be removed in a single step.
- The sharp teeth easily penetrate the cortex.
- The conical opening with the new threads anchors the screw thread securely in the extractor.
- The extractors are 150 mm long and are ideal for removal by puncture incisions.
- Due to the optimal size distribution across 6 different extractors, you will always find the right extractor for the thread diameter of the screws.

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IMPORTANT!

Extractors must always be turned counter-clockwise. Please check the extractors for damage after use. Extractors with damaged threads or teeth after use must be replaced.

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For broken off screws

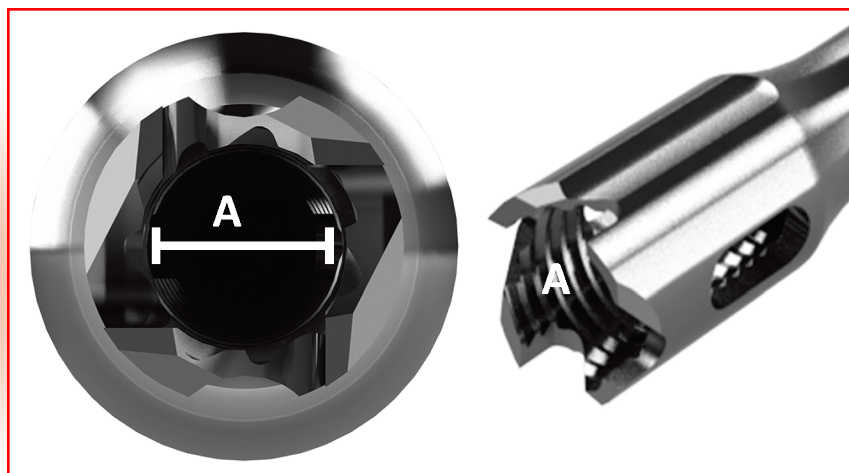
Selecting the right extractor

Correctly selecting the appropriate extractor is of great importance for successful removal of the screws. Each extractor features a size specification that makes the initial selection process easier. The size specifications refer to the diameter of the screw threads of the screws to be removed.

The „AS extractors“ have a conical, tapered internal „left-hand thread.“

The specified dimension „A“ refers to the diameter of the internal thread at the beginning of the extractor.

The dimension of the internal thread „A“ must always be larger than the thread size of the screw to be removed (pay attention to the information in the table on page 7). (Fig.)

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Connect the AS extractor with the drill adapter and the adapter with a drill that has a 3-jaw chuck. Now determine the exact position of the screw in the bone with the help of the image converter. As a rule, the hole through which the screw was originally inserted into the bone can still be detected. Place the AS extractor over the existing hole on the cortex.

For a successful removal of the screw, it is very important that the opening of the extractor is screwed exactly „orthograd“ in the direction of the screw thread.

If all parameters are correct, turn the drill counter-clockwise. The tips of the extractor will first expand the hole of the cortex. Once the internal thread of the extractor has reached the external thread of the screw, the extractor will continue to slide over the screw thread. Turn the extractor over the external thread of the screw until it is firmly anchored to the screw thread. Then pull-out the extractor (without further rotation) of the bone together with the screw.

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