**GIPSSÄGE / AUTOPSIESÄGE
ABSAUGUNG FÜR GIPSSÄGE**

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Any/all content, including, without limitation, all text, illustrations, images, graphics, pictures, organization, structure, and the look and feel of this document also the intellectual property found within in this document is the property of Digimed Medizintechnik and subject to copyright and other applicable laws and also other laws governing the protection of intellectual property. All rights reserved, no responsibility will be taken for the correctness of the information. Errors and alterations excepted. All information and data are subject to change, design and product modifications are reserved. Also we reserve the right of technical changes without prior notice. Particular attention: Please take care that all safety guidelines in the instruction manual, national/international safety rules/ standards and national/international laws be applied. You are responsible for checking, repairing and the putting into operation of all electrical, electronic and mechanical systems! You must make a validation, test and system start-up by your own and at your risk. You are responsible for the observance of statutory and administrative environmental protection regulations/laws, electrical protection regulations/laws, medical protection regulations/ laws and other necessary regulations/laws. Any user is responsible for complying with these regulations and can be held vicariously liable in case of misuse. Revoking of spare parts is not possible. Our company carries out the repair of your apparatus during its guarantee. If you repair it yourself, by a third person or a by third party/company, you lose this guarantee.

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OSCILLATING PLASTER SAW ELEKTRONIC POWER

With this relatively quit plaster saw, you can remove synthetic hard dressings, plaster / white plaster dressings quickly and safely. Due to the stepless speed regulation and the fully automatic load compensation over the entire oscillation range, the plaster sax can be optimally adapted to the required application.

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PLASTER SAW - ELECTRONIC POWER

TECHNICAL DETAILS:

Art.-Nr. 30-200-12	Art.-Nr. 30-200-14	Art.-Nr. 30-210-12	Art.-Nr. 30-210-14
Each of 1 sawblade Ø 50 and Ø 65 mm, (only) for white gypsum plaster	Each of 1 sawblade Ø 50 and Ø 65 mm, for synthetic casts and / or white gypsum plaster	Each of 1 sawblade Ø 50 and Ø 65 mm, (only) for white gypsum plaster	Each of 1 sawblade Ø 50 and Ø 65 mm, for synthetic casts and / or white gypsum plaster
230 V~, 50 Hz, 500 W 3500 – 24000 oscillations, infinitely variable 5 meter cable with Euro-plug 1 pair wrenches for sawblade changing		120 V~, 60 Hz, 500 W 3500 – 24000 oscillations, infinitely variable 3 meter cable with US-plug 1 pair wrenches for sawblade changing	

- 500 Watt power
- 3500 bis 24000 oscillations, infinitely variable
- Full automatically power-compensation above the oscillations range
- Slim motor edition with silent motor ca. 65 – 85 dB(A) ± 5 dB(A) (Level 1 to 3) in relation to the selected oscillations, under unfavourable operating conditions ca. 95 dB(A) ± 5 dB(A) (Level 4 to 6)
- Latest electronic system with various protection and security functions
- Special positioning of the motor, protected by plaster dust
- Robust construction for highest loading capacity by continuous operation
- Safety ON / OFF switch
- Weight only 1,25 kg without cable (1,5 kg with cable)
- Vibrations- and maintenance-free
- Regulation from outside of the manual operating area
- 230 Volt version with 5 meter cable with EU-plug or 110 Volt version with 3 meter cable with US-plug
- Double protection isolation according to class II, medical use of typ BF
- Radio and TV suppressed
- CE conformity
- EN 60601-1, EN 60601-1-2, IEC 60601-1, IEC 60601-1-2 inspected

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- 500 Watt power
- 500 bis 24000 oscillations, infinitely variable
- Full automatically power-compensation above the oscillations range
- Slim motor edition with silent motor ca. 65 – 85 dB(A) ± 5 dB(A) (Level 1 to 3) in relation to the selected oscillations, under unfavourable operating conditions ca. 95 dB(A) ± 5 dB(A) (Level 4 to 6)
- Latest electronic system with various protection and security functions
- Special positioning of the motor
- Robust construction for highest loading capacity by continuous operation
- Safety ON / OFF switch
- Weight only 1,4 kg without cable (1,6 kg with cable)
- Vibrations- and maintenance-free
- Regulation from outside of the manual operating area
- 230 Volt version with 5 meter cable with EU-plug or 110 Volt version with 3 meter cable with US-plug
- Double protection isolation according to class II, medical use of typ BF
- CE conformity
- Tested and conform with IEC CISPR 11:2005 IEC 60601-1:2005+AMD1:2012 IEC 60601-1-2:2014 IEC 60601-1-6:2010+AMD1:2013 IEC 61000-3-2:2014 IEC 61000-3-3:2013+AMD1:2017 IEC 61000-4-28:1999+AMD1:2001+A MD2:2009
- National differences EU Group differences EU Special National Conditions EU A-Deviations for CA (Canada) and US (United Staates of America)

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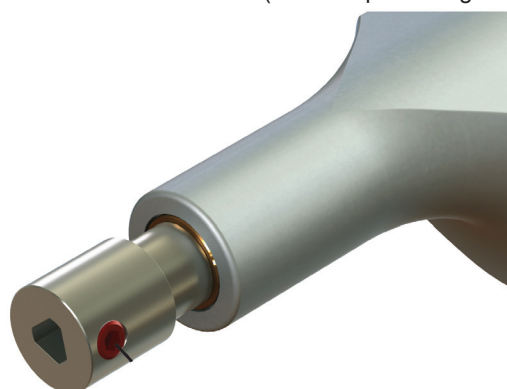
AUTOPSY SAW EPA3K - ELECTRONIC POWER

TECHNICAL DETAILS:

Set Art. No. 30-250-00	Set Art.-Nr. 30-255-00
inclusive (comes along with): 1 x Autopsy Saw EPA3K, 230V,50Hz,500W (Art.-No. 30-220-01) 1 x Art.Nr. 30-260-10 Sawblade Ø50 mm 1 x Art.Nr. 30-260-13 Sawblade Ø65 mm 1 x Art.Nr. 30-260-30 Segment sawblade 50/65 mm 2 x Art.Nr. 30-198-03 Wrenches 14 mm 1 x Art.Nr. 30-199-00 Hexagon key with T-grip 1 x Art.Nr. 30-200-02 Grease (filled in a syringe)	inclusive (comes along with): 1 x Autopsy Saw EPA3K, 120V,60Hz,500W (Art.-No. 30-225-01) 1 x Art.Nr. 30-260-10 Sawblade Ø50 mm 1 x Art.Nr. 30-260-13 Sawblade Ø65 mm 1 x Art.Nr. 30-260-30 Segment sawblade 50/65 mm 2 x Art.Nr. 30-198-03 Wrenches 14 mm 1 x Art.Nr. 30-199-00 Hexagon key with T-grip 1 x Art.Nr. 30-200-02 Grease (filled in a syringe)
Option Art.Nr. 30-130-32 Cable cord AUSTRALIA Cable cord with Euro-plug will be replaced to a cord with Australian plug. Only available for 230 Volt version. Cable cord length ca. 3 meters.	120 Volt, 60 Hz cable cord 3 meters with US-plug WITHOUT any triangular arbor
Option Art.Nr. 30-130-33 Cable cord GREAT BRITAIN Cable cord with Euro-plug will be replaced to a cord with Great Britain plug (BS 1363). Only available for 230 Volt version. Cable cord length ca. 3 meters.	
Option Art.Nr. 30-130-40 Plug GREAT BRITAIN Euro-plug will be replaced to a Great Britain plug (BS 1363). Only available for 230 Volt version.	
230 Volt, 50 Hz cable cord 5 meters with Euro-plug WITHOUT any triangular arbor	

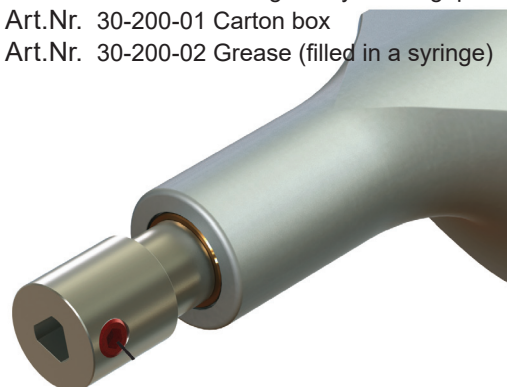
REPLACEMENT EQUIPMENT

- Art.Nr. 30-101-00 Allen setscrew
Only for 30-250-00 or 30-255-00
- Art.Nr. 30-130-20 Carbon brushes, 230 Volt
- Art.Nr. 30-130-30 Cable cord EURO
Cord with Euro-plug. Cord length ca. 5 meters.
- Art.Nr. 30-130-32 Cable cord AUSTRALIA
Cord with Australian plug. Cord length ca. 3 meters.
- Art.Nr. 30-130-33 Cable cord GREAT BRITAIN
Cord with Great Britain plug (BS1363).
Cord length ca. 3 meters.
- Art.Nr. 30-140-30 Plug GREAT BRITAIN
Great Britain plug BS 1363
- Art.Nr. 30-140-20 Carbon brushes, 120 Volt
- Art.Nr. 30-140-30 Cable cord USA
Cord with USA plug. Cord length ca. 3 meters.
- Art.Nr. 30-198-03 Wrenches 14 mm, pair
- Art.Nr. 30-199-00 Hexagon key with T-grip
- Art.Nr. 30-200-01 Carton box
- Art.Nr. 30-200-02 Grease (filled in a syringe)


TECHNICAL SPECIFICATIONS
(30-250-00 OR 30-255-00)

- 500 Watt Power
- 6500 - 24000 oscillations, infinitely variable
- Slim motor edition with silent motor 52 - 77 dB(A), in relation to the selected oscillations
- Latest electronic system
- Various protection and security functions
- Robust construction for highest loading capacity by continuous operation
- Safety ON / OFF switch with soft start
- Weight only 1,6 kg (1,3 kg without cable)
- Less vibrations
- Maintenance-free
- Regulation outside operating area
- Double protection isolation according to class II
- Applied part type BF
- CE conformity according Directive 93/42 EEC, Annex VII
- CE classification according to class 1, rule 12
- EN 60601-1, EN 60601-1-2, EN 60601-1-6

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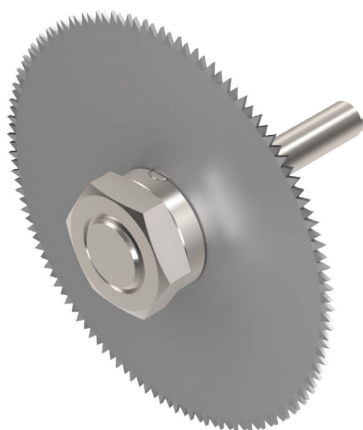
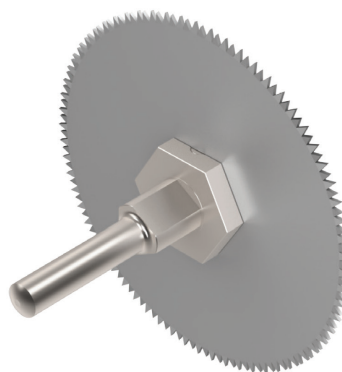
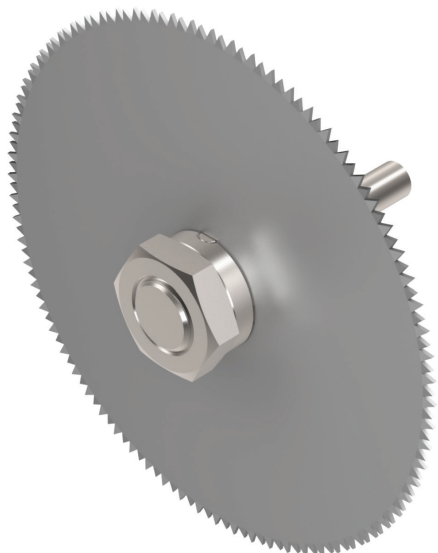
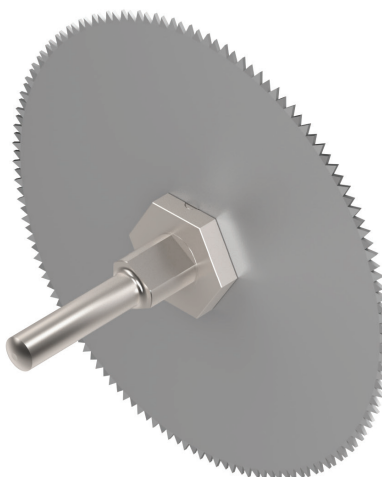
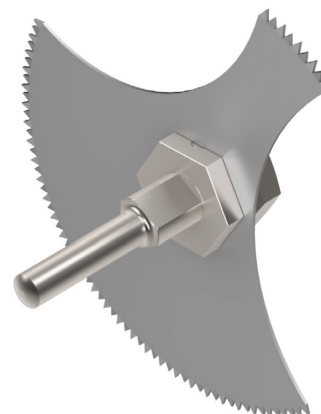
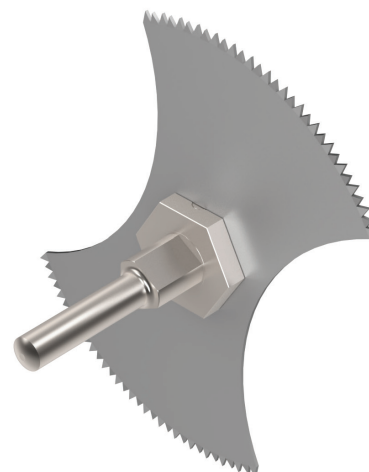
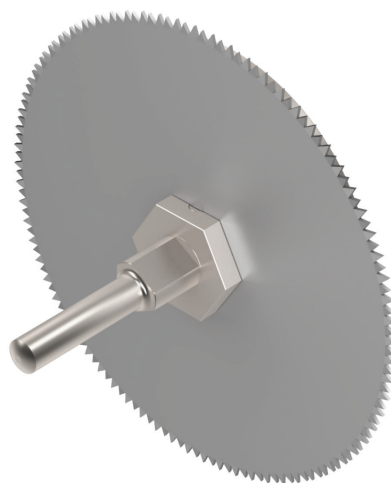
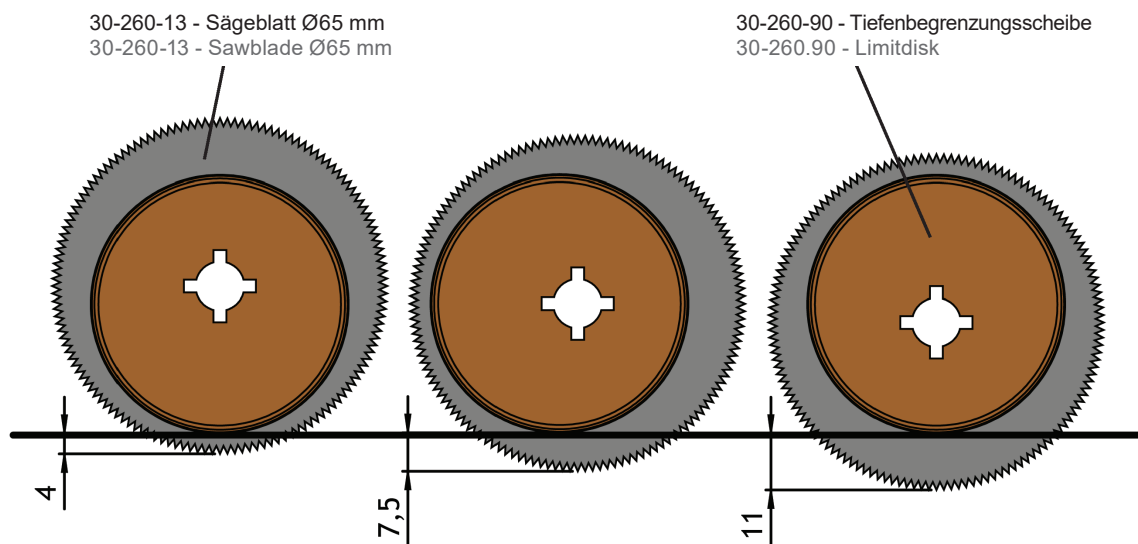
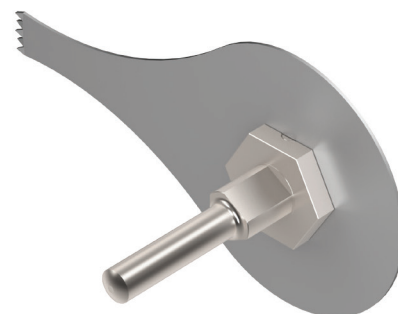
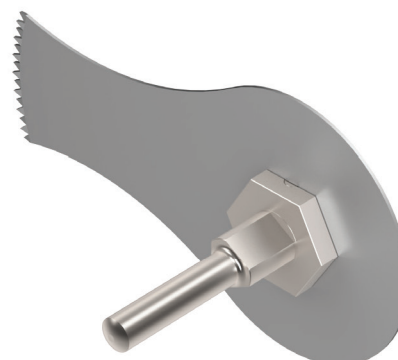
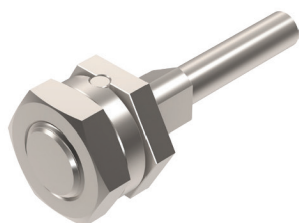
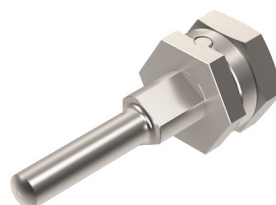
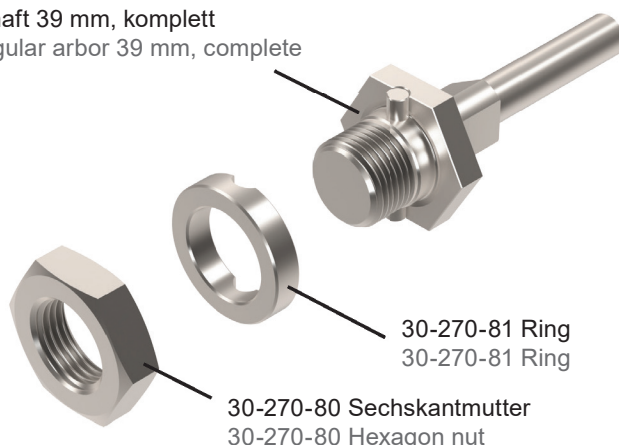
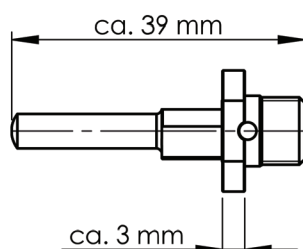
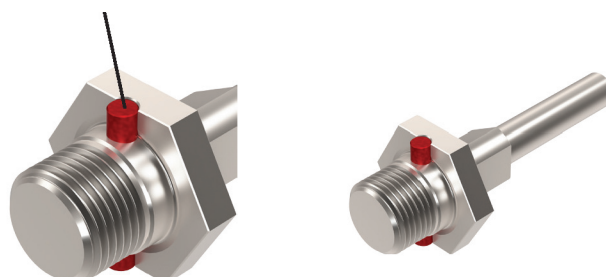
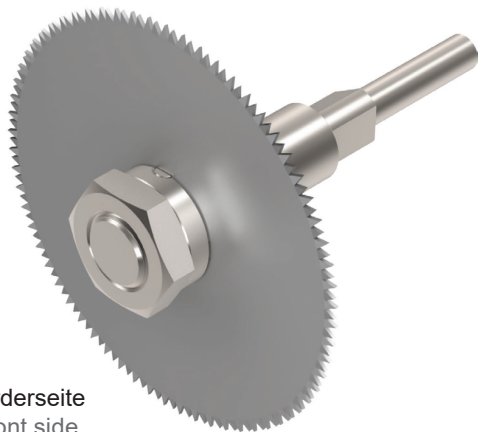
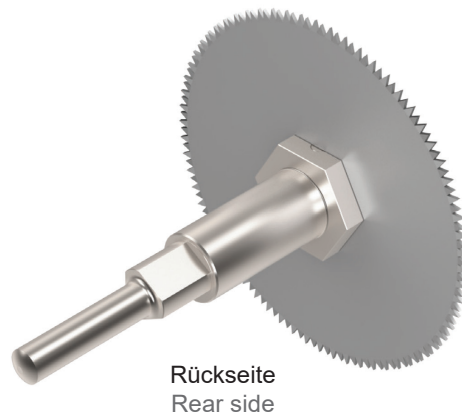
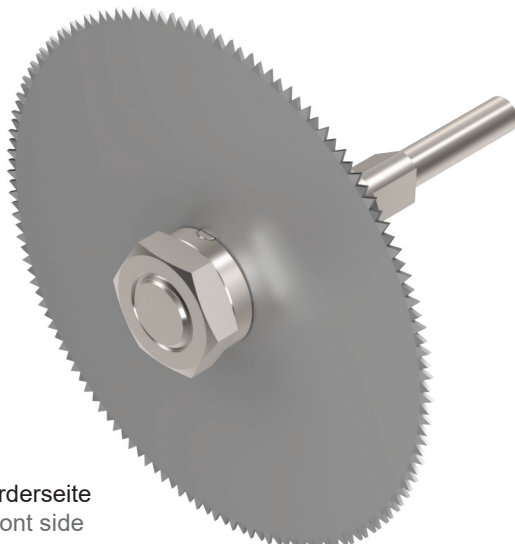
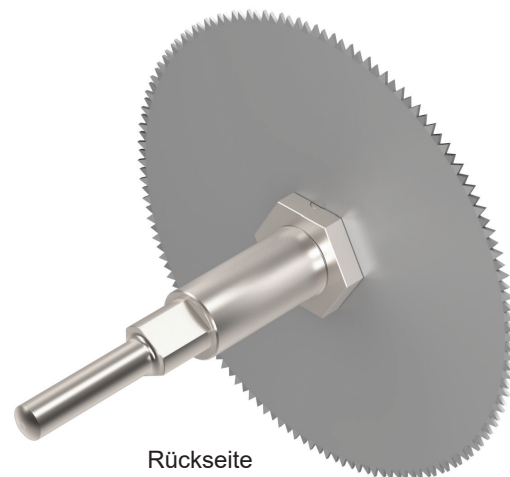
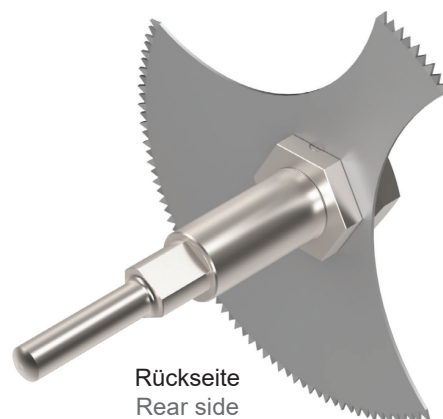
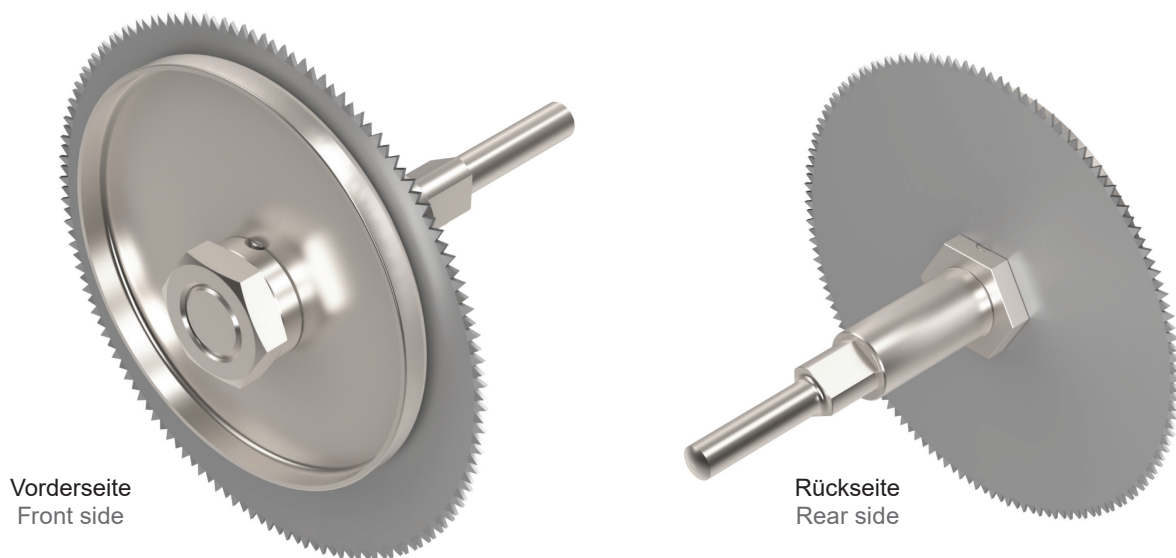
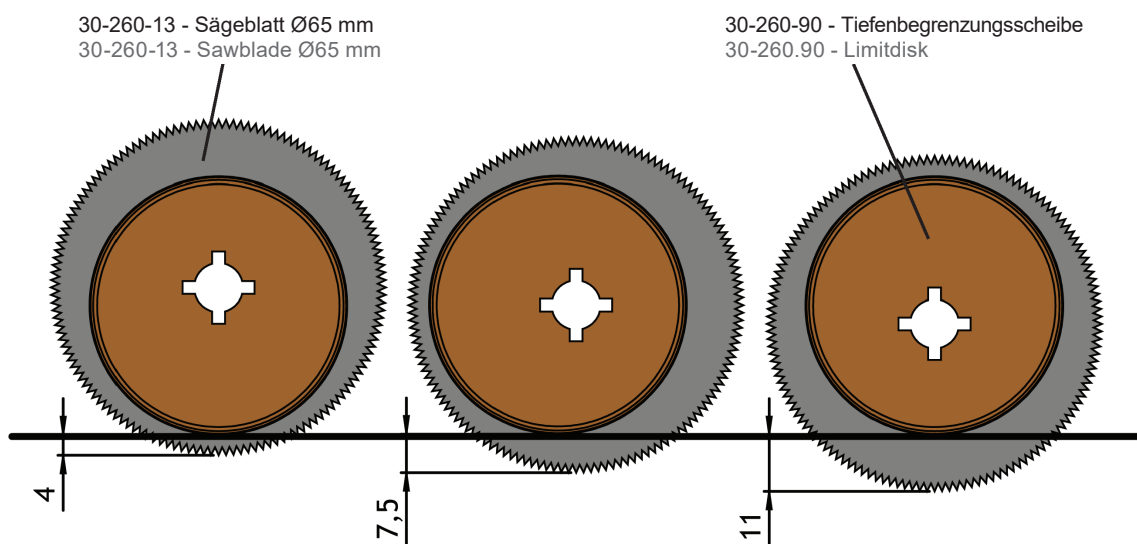
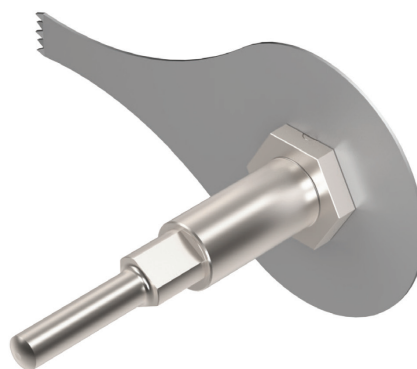
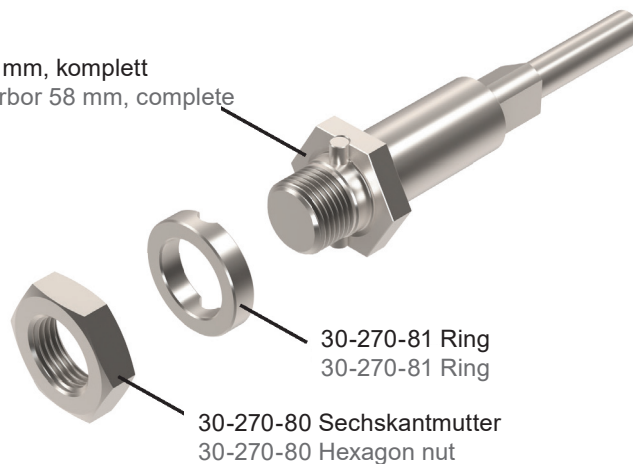
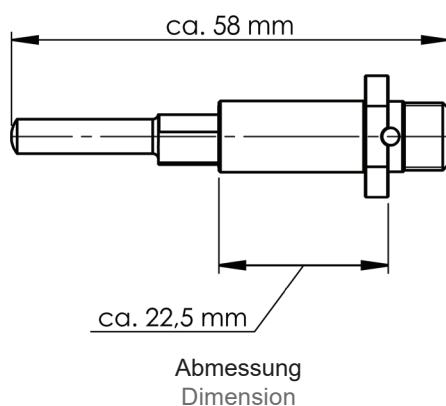
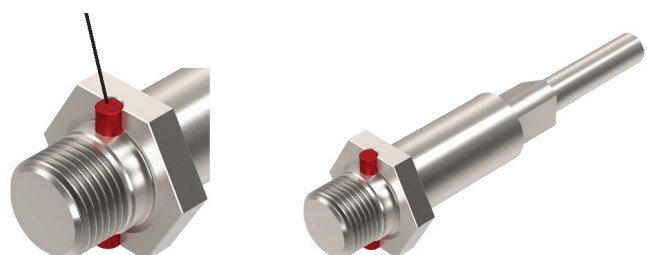
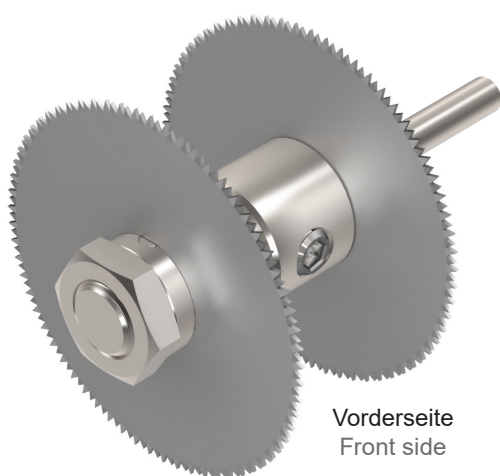
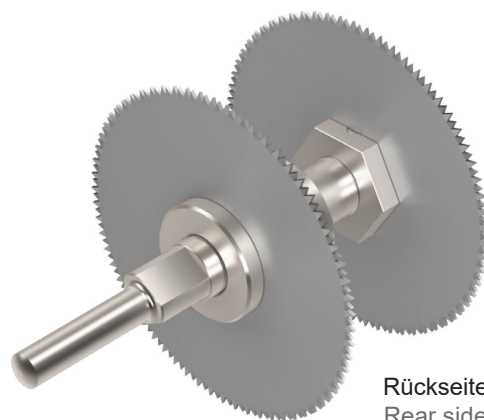
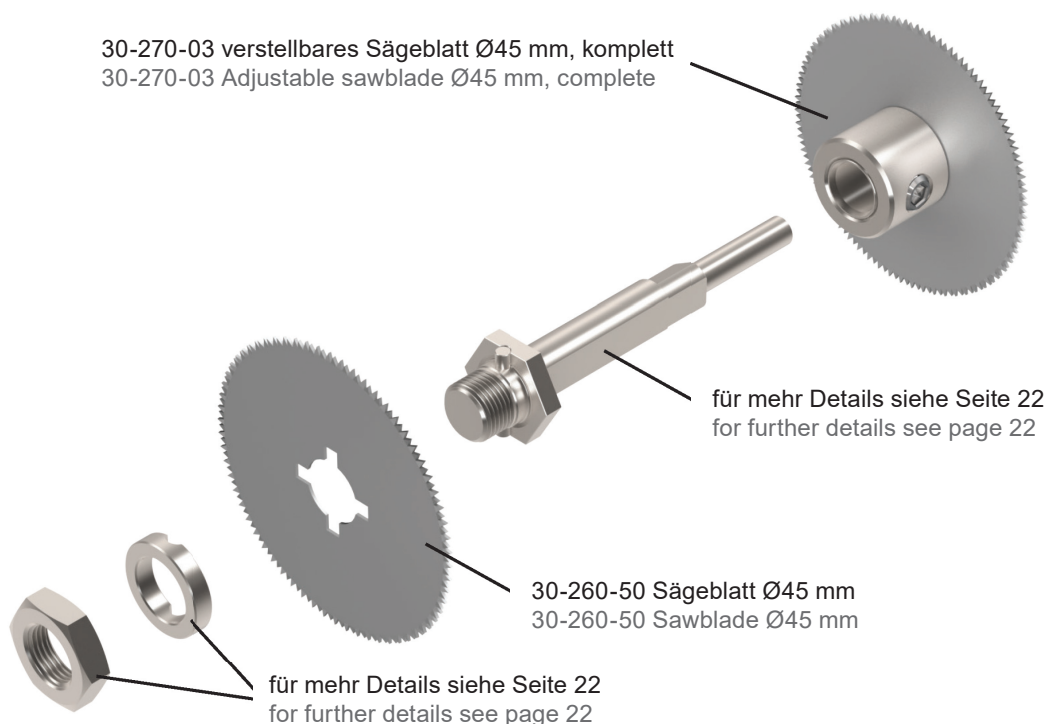
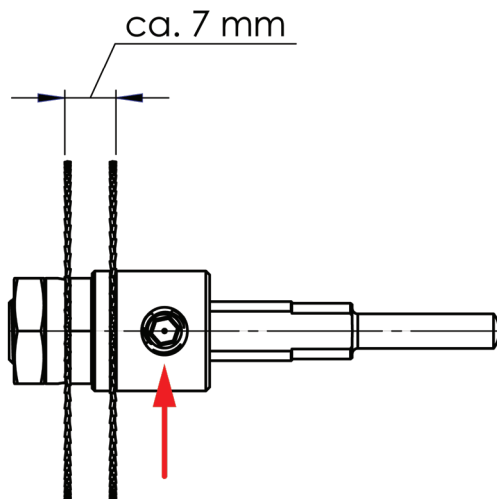
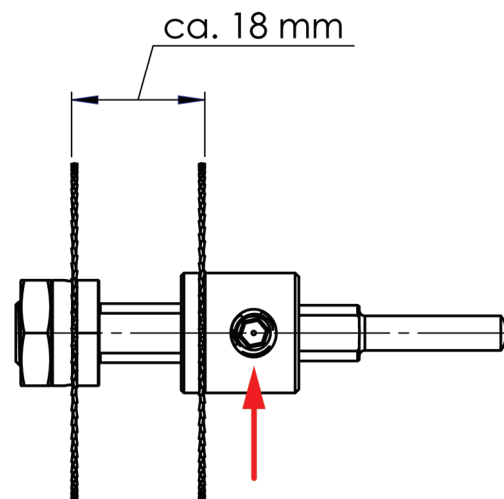
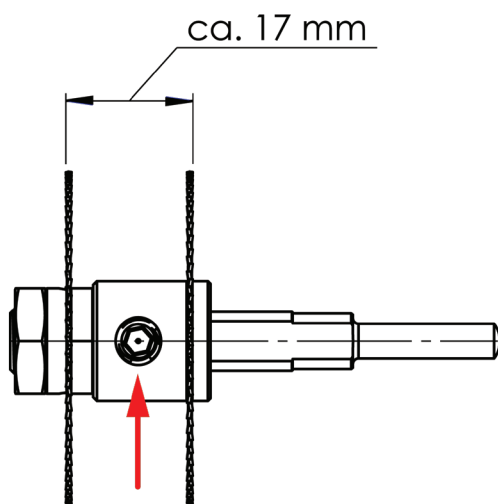
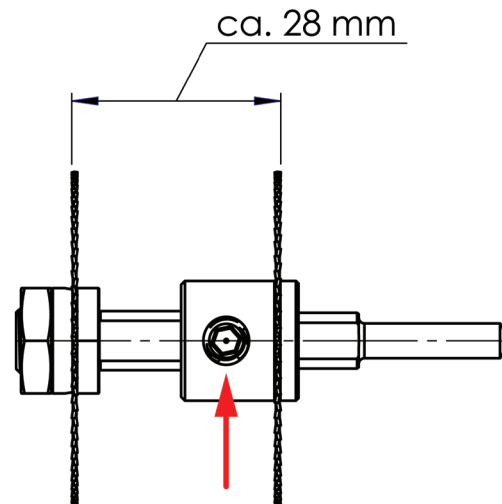
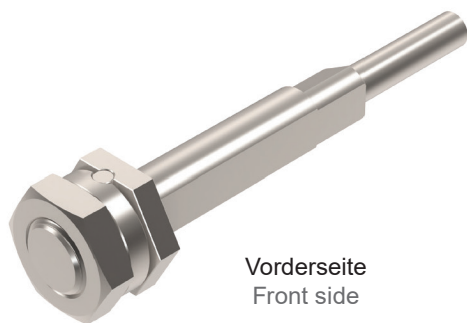
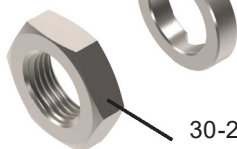
30-270-81 Ring
30-270-81 Ring


30-270-80 Sechskantmutter
30-270-80 Hexagon nut

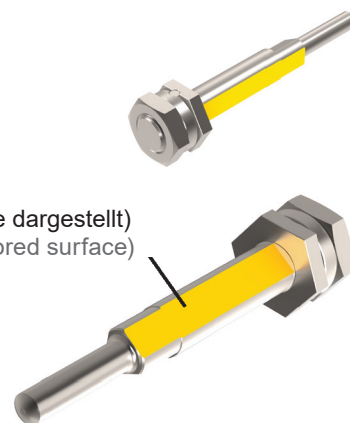Ebene Oberfläche (als Gelbe Fläche dargestellt)
Plane surface (shown as yellow colored surface)



Art. No. 30-270-70	3k-Schaft Doppel Säge, allein Triangular arbor double saw, alone
Ersatzteil nur verwendbar mit der 3k-Schaft Doppel Säge, allein (Artikel-Nr. 30-270-70) spare part only usable with the Triangular arbor double saw , alone (article no. 30-270-70)	

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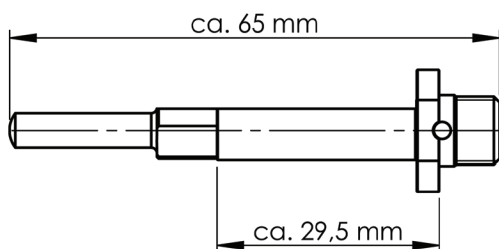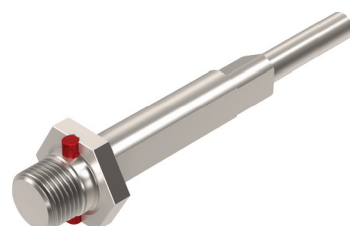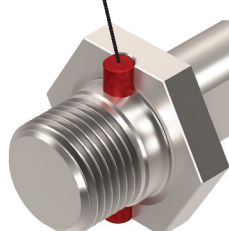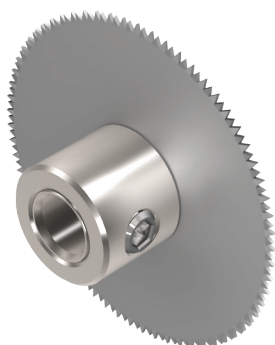
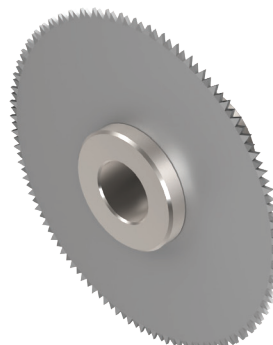
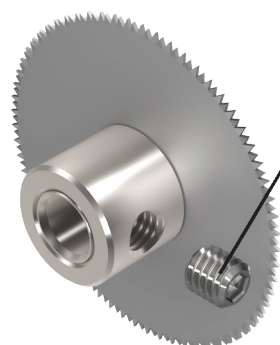
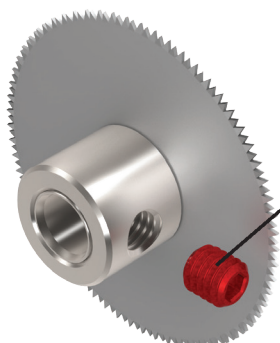
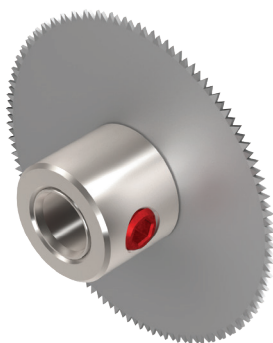
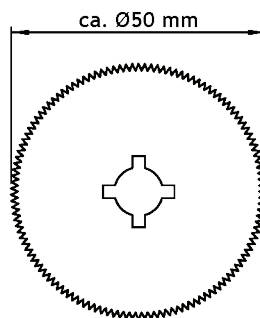
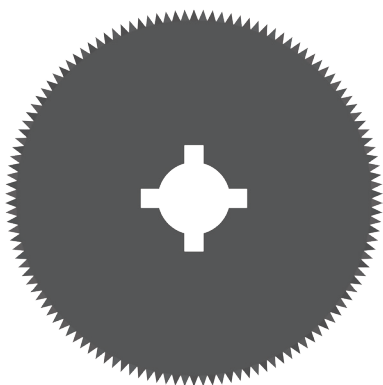
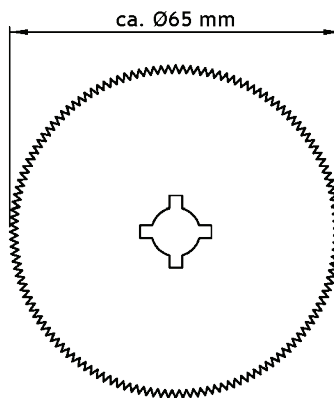
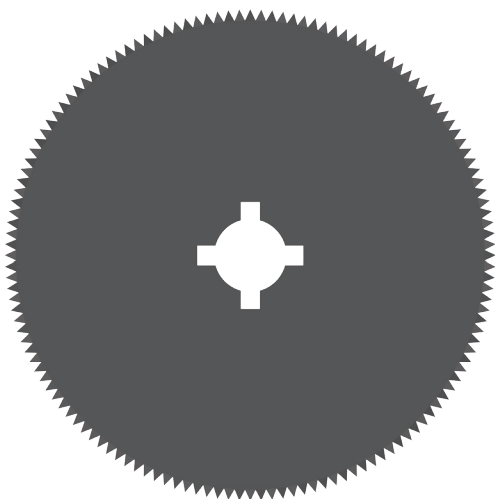
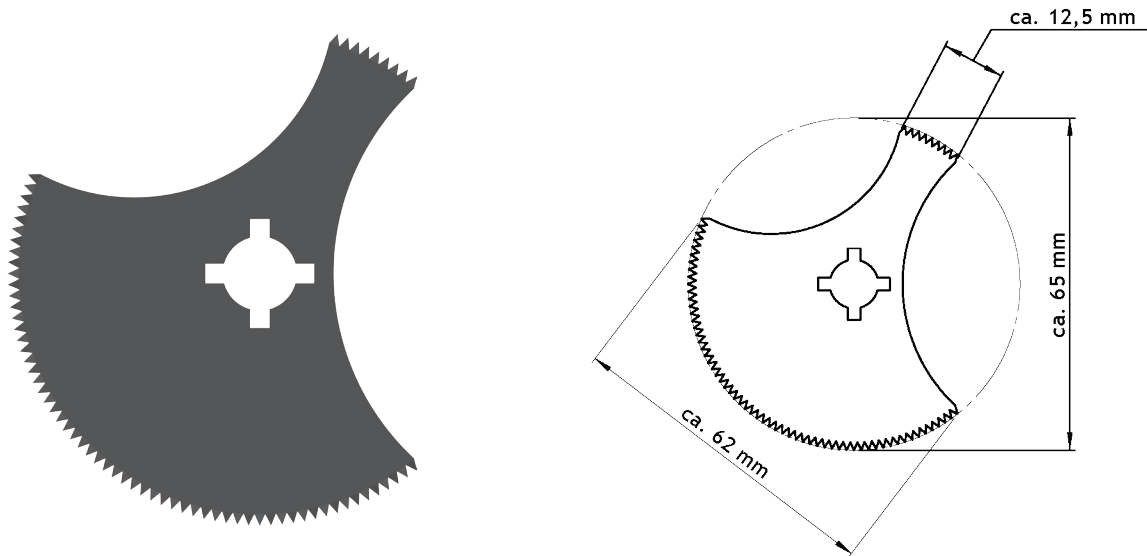
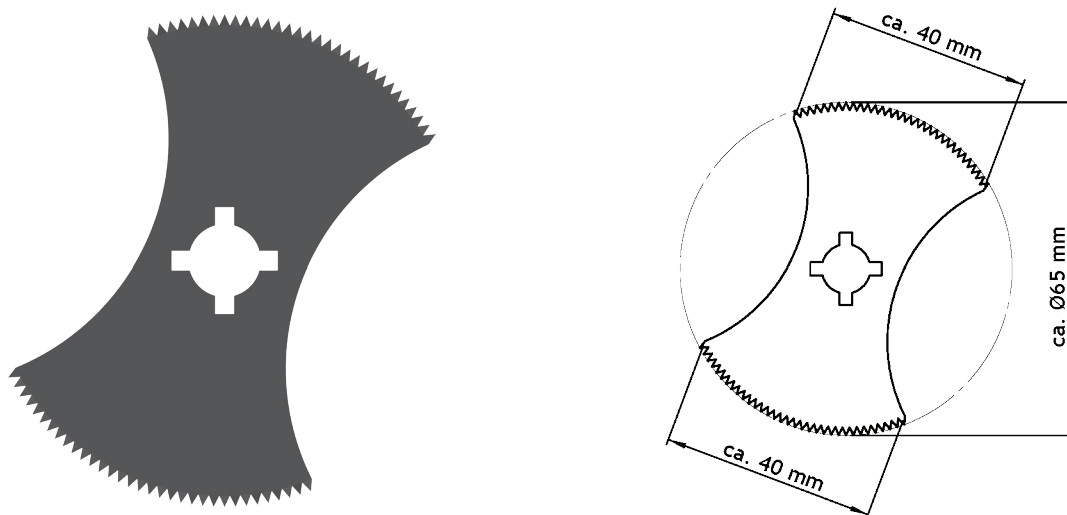
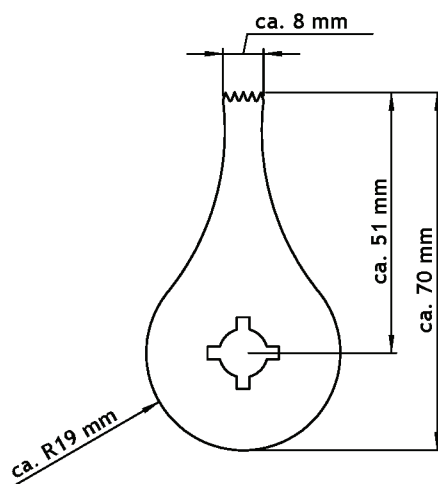
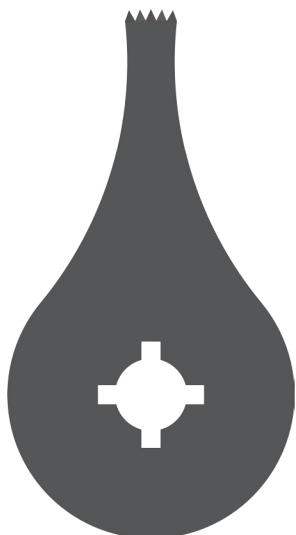
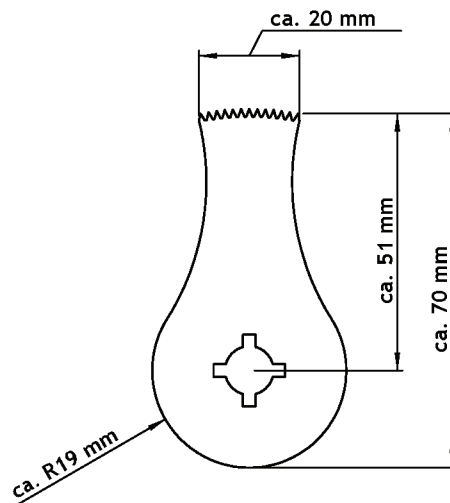
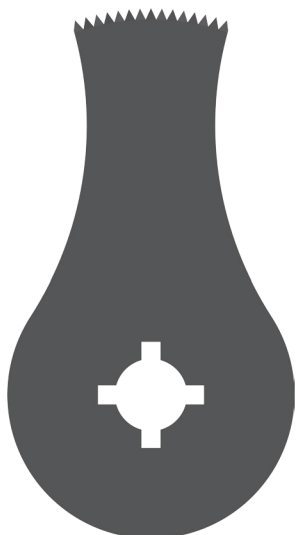
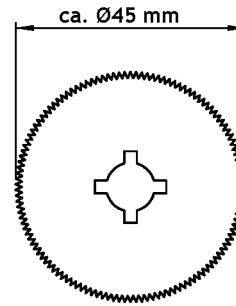
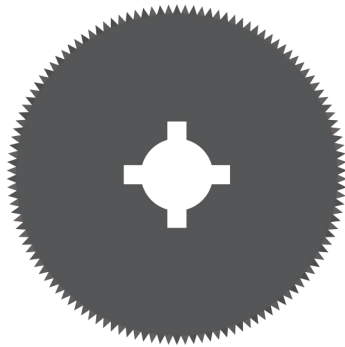
Wechselstrom

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AUTOPSY SAW EPA / EPA3K „ELECTRONIC POWER“ INSTRUCTION MANUAL

The autopsy saw „Electronic Power“ is an oscillating saw for using in pathology departments for autopsy (post-mortem).

For proper changing of the saw blades please use the attached wrenches. It is important that the excenter is not overstrained. In order to avoid wear and tear as well as any heating of the saw, we highly recommend replacing worn and blunt saw blades in time. Please be aware, that saw blades have sharp cutting edges. This has to be considered during use as well as when replacing them. Caution of injuries!

After use any dust, blood or similar should be wiped off the saw and the blade with a dry cloth. The saw should not be soaked in any liquids (like water etc.) or be washed or even covered on. If disinfection is required, it has to be carried out by using appropriate diluted disinfecting solutions.

The saw should be only wiped by a damp cloth (do not cover it).

It is important that no liquids or steam enter the equipment. Please unplug the saw before cleaning.

The saw (complete with motor) should not be sterilized. Only saw head (driving gear) is to be treated by autoclave with steam only! Do not treat electric motor by autoclave, avoid penetration of water and other liquids and/or damps!

DISMANTLING: Remove hexagon head screw, safety loop and saw blade. Loosen the 2 hexagon socket head cap screws. Remove saw head (driving gear) of the electric autopsy saw and treat only by autoclave.

ASSEMBLAGE: Effected in the other way around. It is very important to grease the oscillating crank well before assembling. The saw is maintenance-free. The saw should not be used in areas with danger of explosion.

Before using the saw, it ought to be checked for any mechanical damage. Furthermore, the saw should not be used and has to be sent for a supervision, if it has been dropped onto the floor. Please do not tempt to repair it, as this will void your warranty and additional defects may occur.

In order to assure proper disposal, the autopsy saw can be disposed by us or third parties.

Blunt saw blades (clean and sterilized) have to be disposed off by using the recycling process (steel-/ metal raw materials).

The autopsy saw is switched off/on by a security switch. By switching on the saw a high running-up electricity is effected for a short time. The electronic system avoids that the fuses will adversely react (limitation of the running-up electricity). After the warming-up to the selected frequency of oscillations, the full power of the saw can be used. If you start your saw carefully, all mechanical and electrical parts will be protected. The risk of injuries is reduced by a smooth starting.

The security switch assures that at no time the autopsy saw will be overstrained. In case of overloading, e.g. by blocking the drive, the motor will be switched off immediately. Thanks to the security switch, the saw can be used after a short break.

The autopsy saw is developed in accordance with EMV norm EN 60601-1-2.

The regulation of the motor performance is on the rear of the motor (close to the cable connection) and therefore won't disturb the user during use. At the control button the frequency of oscillations required can be infinitely variably selected from 6500 up to 24000. Once the selected, the oscillations will remain constant, if the motor is in neutral or during the cutting.

Autopsy saw, standard

Autopsy saw, special tringle chuck

AUTOPSY SAW EPA / EPA3K „ELECTRONIC POWER“ INSTRUCTION MANUAL

The load compensation is permanently active and regulates the oscillations, even if various loads are occurring. The Speedo on the drive shaft registers a reduction of the oscillations, if the burden of the motor is increased. By comparing the selected power and the number of required oscillations, the operation of the system will be adjusted permanently. In case of a reduced number of oscillations due to an excess load, the electronic system will re-adjust the frequency of oscillations.

This procedure is done so quickly, that the user won't recognize any difference. Therefore the plaster saw offers an optimised number of oscillations during use. In case of excessive use of the motor (e.g. if saw blades are stuck) the electronic system switches off the saw. The saw can be re-used after a short break.

By using the latest electronic system and electrical parts we are able to offer you a slim motor with a user friendly housing. Furthermore ergonomic and anthroposophic aspects have been considered in respect of smaller hands, noiseless during use (as not to frighten children while cutting the plaster, we recommend a lower number of oscillations). The ambiental temperature for storing and transportation should not fall below 5 °C and not exceed 40°C. Furthermore humidity should not exceed 80%.

TECHNICAL DATA / CLASSIFICATION

Part.-Nr. 30-210-01, autopsy saw, standard = 230 V, 50 Hz, 500 W

Part.-Nr. 30-215-01, autopsy saw, standard = 110 V, 60 Hz, 500 W

Part.-Nr. 30-220-01, autopsy saw, special triangle chuck = 230 V, 50 Hz, 500 W

Part.-Nr. 30-225-01, autopsy saw, special triangle chuck = 110 V, 60 Hz, 500 W

The equipment is in accordance with classe II and typ BF.

TECHNICAL SPECIFICATION

- 230 V, 50 Hz, 500 W or
- 110 V, 60 Hz, 500 W
- infinitely variable from 6500 up to 24000 oscillations
- regulation from outside of the manual operating area
- automatic load compensation for the whole oscillations amplitude
- latest electronic system with various protection and security functions
- 5 m cable with EU-connection, 230 V
- 3 m cable with US-connection, 110 V
- special positioning of the motor, protected by plaster dust
- reinforced armature shaft
- slim motor edition with silent motor 52 – 77 dB(A) in relation to the selected number of oscillations

SYMBOL

 double protection isolation according to class II

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Airflow 60 liter / second


Vacuum „Exclusiv“

Technical details / specifications see overleaf

Pictures shows the plaster saw
Electronic Power also available
with the plaster saw Blue Power

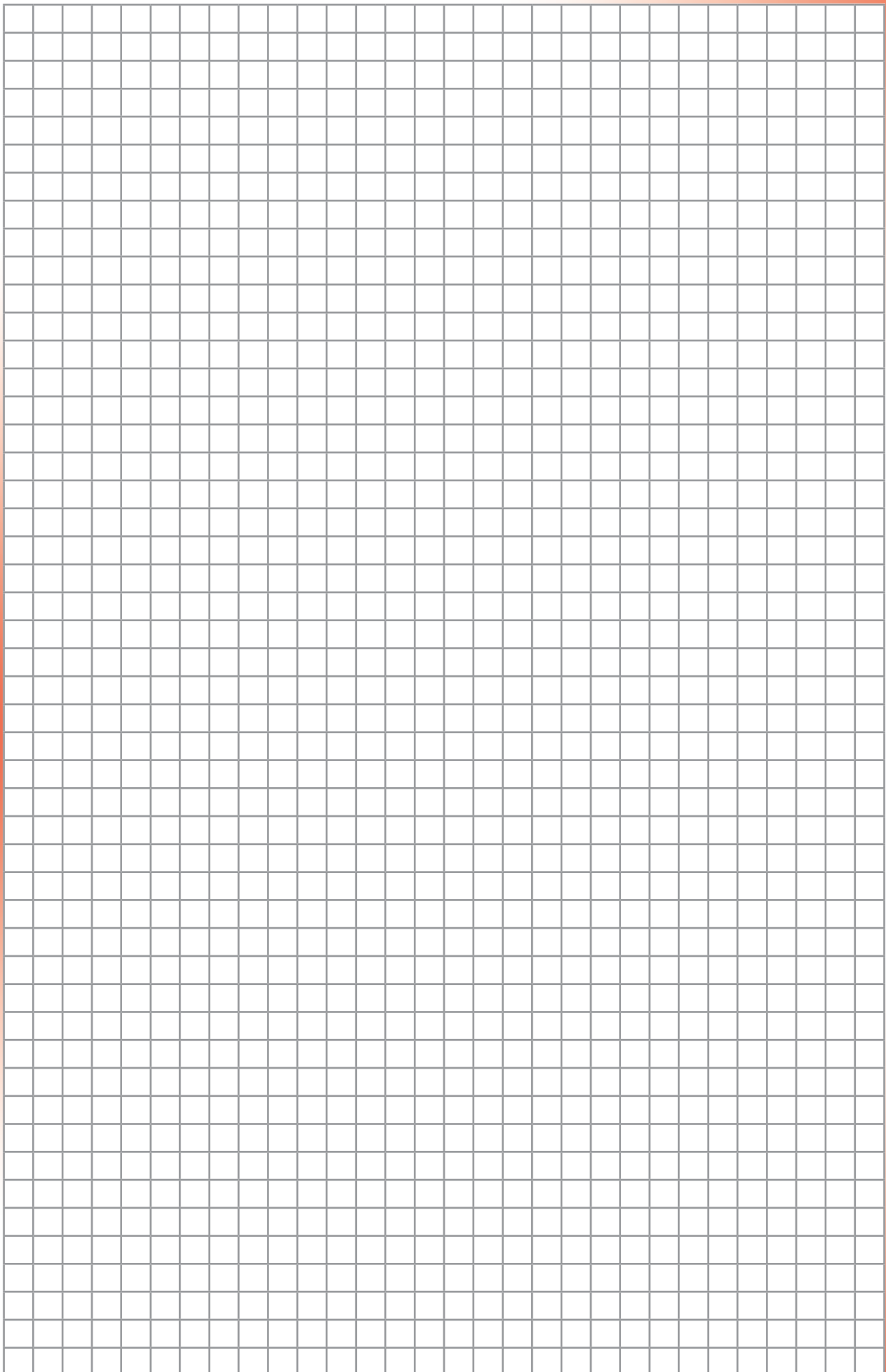- ✓ **S-Class HEPA-Filter, HEPA class H11, antiseptic treatment, 99,97% of all particle who are $\geq 0,3$ mikron are filtered**
- ✓ **Automatically turn on / off from the cleaner with the turn on / off from the plaster saw**
- ✓ **By the turn off from the plaster saw the cleaner will going off after 6 to 10 sec. later to clean the hole system**
- ✓ **Robust construction for highest loading capacity by continuous operation**
- ✓ **Elektronik power control vacuum motor**
- ✓ **Noise level vacuum only, 63 – 77 dB(A)**
- ✓ **Antistatic suction hose**
- ✓ **Dust bag, 3-layers, capacity approx. 14 liter**
- ✓ **Control lamp for full dust bag**
- ✓ **“Exclusiv” model : cleanertube is in the pipe and not on the floor**
- ✓ **“Exclusiv” model : better handling**

Digimed Medizintechnik

Inhaber: Patrick Mohr

Kreutzerstrasse 1

78573 Wurmlingen

Telefon: +49-(0)7461-9101172

Fax: +49-(0)7461-9101172

info@digi-med-de

www-digi-med-de

Rev.: 2024